



Sports Brace

Sports Brace—Empowering People

Quickie Knee Immobilizer

HCPCS: L1830

SIZE: Universal



FEATURES:

- Universal sizing will fit most patients
- Elastic Velcro straps allow for added compression
- Two movable stays for medial and lateral placement
- Contoured posterior stays for extra rigidity

INDICATIONS: Sprains, strains of the knee ACL, PCL, MCL, LCL injuries.

HOW TO APPLY / CARE INSTRUCTIONS: Practical knee immobilizer is quick and easy to fit and can be cut to fit a range of sizes. Place behind knee using the contoured posterior stays as a guide. The inverted portion of the stays should be placed in the popliteal area (directly behind the knee). Position the movable medial and lateral stays so that they are aligned with the side of the leg. Wrap the sides of the immobilizer around and close in front. Cut the side without hook allowing for 2" of overlap. Support should fit snug but not so tight to restrict circulation. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat.

Visit us online at www.sportsbrace.net