

Deluxe Universal Arm Sling

HCPCS: A4565 SIZE: Universal



FEATURES:

- Envelope style
- · Can be folded to proper length to accommodate most patients
- · Soft foam neck pad for greater patient comfort
- Slide buckle on strap for easy adjustment
- Velcro closure

INDICATIONS: Arm injuries or instabilities. Post-surgery.

HOW TO APPLY/CARE INSTRUCTIONS: Unfasten hook and loop closure on strap. Insert arm into the pouch with elbow in the pouch corner. If needed, fold-in the sling to shorten the length and secure the hook tab on the loop closure. Bring strap across the back and over the opposite shoulder. Run strap through round loop; adjust to desired length. Press hook and loop closure against the strap to secure. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat.

Visit us online at www.sportsbrace.net