



Sports Brace

Sports Brace—Empowering People

UNIVERSAL WRIST BRACE W/THUMB SPICA

HCPCS Code: L3807 and L3809

SIZE: Universal Left and Right



FEATURES:

- Constructed of durable, lightweight material
- Universal sizing
- Adjustable dorsal stay
- Removable and malleable aluminum, palmer stay for correct fitting

INDICATIONS: Wrist immobilization, De Quervain's Syndrome, wrist and thumb sprains and strains, Carpal tunnel syndrome, tendonitis, and post cast healing.

HOW TO APPLY / CARE INSTRUCTIONS: Unfasten straps and slide affected hand into brace. Position palm stay firmly against the palm of the hand. Ensure the edge of the brace is below the knuckles. Detach and align dorsal stay with middle knuckle. Reattach. Pull and wrap the lacing straps around wrist. Secure when the desired fit and level of compression is reached. Fasten Closures, Hand Wash Cold, Use Mild Soap Only, Air dry away from heat.

Visit us online at www.sportsbrace.net